Airon Polyester Strip

ENGLISH

Presentation:

Envelope with 50 / 100 units.

Description:

The Airon polyester strip has as an objective to confine, under pressure, the restorative material in the dental cavity, preventing a formation of material excess in the interproximal space.

Indication:

Suitable for restoration procedures.

Composition:

Polyester.

Directions of Use:

1. Use the matrix holder and wooden wedge to fix the strip;
2. Pull the strip on the rounded end of clinical tweezers to give the desired format;
3. If necessary, make an indentation in length and width;
4. Next, the matrix should be taken between the teeth, so that the convex area is properly located.

* The material that can be used in conjunction with Airon Polyester Strips during the restoration procedure are the Acid Gel 37%, resin composite, cements and adhesives.

Warning and Precautions:

- The use is restricted to professionals specialized in dentistry, according to local legislation and with specific training for its handling.
- Procedures provided in the directions of use should be strictly followed
- Product of single use should be discarded after use.
- Handle the product carefully in order to avoid further injuries to soft tissues in the oral cavity of the patient.
- Do not use the product after the term of validity.
- Keep away from the reach of children.

Contraindications and Side Effects:

Patients with sensitivity to polyester should not use the product.

Storage and Transport:

Store in a clean area, sheltered from the direct sunlight and without moisture.

Disposal:

The product should be disposed of, according to current legislation of its country. The envelope of the product is recyclable and should be discarded in an appropriate manner for such purpose.

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