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**EN Liquid for root canals rinsing**

**PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE**  
TO BE USED BY DENTIST ONLY

**COMPOSITION:** 40% citric acid solution.

**INTENDED USE**

The product is used during endodontic treatment for rinsing root canals. It improves mechanical cleaning of the root canal.

It removes smear layer from the walls of root canal exposing the dental tubules. It allows for precise cleansing of dental tubules from pulp debris and precise sealant penetration.

Adapter with a plug for taking the liquid preparations into the syringe

1. Unscrew the bottle cap.

2. Screw the adapter on the bottle.

3. In order to use the bottle, remove the blue plug, screw the end of the syringe into the adapter opening, tilt the bottle with the preparation upside down and take the liquid into the syringe, while simultaneously holding down the adapter.

Next, put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with a rubber plunger.

4. After each use, close the adapter opening with a blue plug.

5. For transport, remove the adapter and screw the cap on the bottle.

6. Instructions for use

Insert the product into the root canal using the syringe with a needle with a side opening while mechanically preparing the canal. After using the product, rinse the canal with physiological saline.

**CONTRAINDICATIONS**

Do not use the product for patients with hypersensitivity to the product's components. Do not leave the product in the canal or the tooth cavity between visits. This may result in excessive softening of dentin.

**PRECAUTIONS**

Special care should be taken when the product is applied into perforations. It irritates the eyes and skin. In case of contact, immediately rinse with plenty of water and seek medical attention. Use rubber-dam while working with the product. Use protective gloves, glasses and clothing.

Take special care when using this product to children because of the danger of excessive softening of tooth hard tissues.

**DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!**

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

**STORAGE**

Store in closed original packaging below at room temperature 25°C. Keep out of reach of children.

**SHAKE BEFORE USE**

Expiry date is placed on the main container. Once the container is open, the expiry date doesn't change if it the container is securely closed after each use.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

Bottle of 200 g or 400 g of preparation, adapter with a screw cap, dispenser.

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