Description

OptraSculpt Pad is a modelling instrument designed for the intraoral adaptation, shaping and contouring of uncured packable composite filling materials. OptraSculpt Pad consists of an autoclavable handheld instrument and single-use disc-shaped attachments (pads) in diameters ranging from 4 to 6 mm.

The handheld instrument contains a metal spatula on one working end and a holding ring on the other. The single-use disc-shaped attachments, which are made of soft, highly flexible synthetic foam material, are inserted into the holding ring. These OptraSculpt Pad attachments allow users to contour packable composite restorations quickly and effectively without encountering stickiness between the instrument and composite. This presents a clear advantage, particularly in the fabrication of large Class III, IV and V restorations and direct laminate veneers. Reference scales on the handheld instrument assist in establishing appropriate symmetries, tooth width proportions and alignments when creating a restoration with composite materials.

Composition

- The OptraSculpt Pad attachments consist of polyethylene and EVA.
- The handheld instrument consists of stainless steel and PEEK.

Indication

Suitable for adapting, shaping and contouring uncured packable composite and compomer materials in Class III, IV and V restorations and direct laminate veneers.

Contraindication

The use of OptraSculpt Pad is contraindicated

- if the stipulated working technique cannot be applied;
- if a patient is known to be allergic to any of the components of OptraSculpt Pad.

Side effects

None known to date

Application

- NOTE – Brush NEW instruments with soapy water, rinse and dry. Then, carry out a complete hygiene cycle (see "Disinfection, cleaning and sterilization").
- Make sure to use only OptraSculpt Pad attachments and instruments that are in impeccable mechanical and hygienic conditions.
- Make sure to insert the OptraSculpt Pad attachments into the instrument correctly to achieve a tight fit.
- Do not use damaged instruments and attachments.

Select either a 4 mm or 6 mm OptraSculpt Pad attachment, depending on the size of the restoration. Using sterilized tweezers, remove the attachment from the packaging and place it flat on a firm, contamination-free working surface. Next, slide the holding ring over the body of the attachment, applying pressure, until the ring sits flat on the basis of the attachment.

Place the OptraSculpt Pad attachment flat on a working surface.

Slide the instrument over the body of the attachment, applying pressure.

Insert the ring over the body until it meets the base.

OptraSculpt® Pad

EN Instructions for Use

– Composite Sculpting & Modelling instrument

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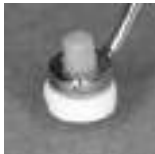
OptraSculpt Pad attachment is correctly inserted in the holding ring.

The OptraSculpt Pad attachments are designed for single use exclusively!

Use the metal spatula on the other side of the instrument to pick up and place the restorative material. After placing the uncured packable composite into the prepared cavity, adapt and contour the material with the soft OptraSculpt Pad attachment, using a dabbing motion.

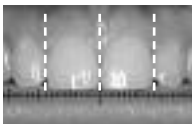
OptraSculpt Pad during clinical use. Use the pad with a dabbing motion, applying only gentle pressure. Place material and remove excess with the metal spatula on the other end of the instrument.

After having completed the restoration, remove the OptraSculpt Pad from the instrument and dispose of it appropriately. Subsequently, disinfect, clean and sterilize the instrument.

Using the reference scales


The OptraSculpt Pad instrument features two reference scales.

Scale 1 provides plain reference markings to compare the tooth-to-tooth width proportions. In addition, the horizontal line on the scale facilitates the evaluation of the incisal edge line.

Scale 1 can be aligned with the incisal edge to e.g. compare the contralateral tooth width proportions and/or evaluate the incisal edge line against the horizontal line on the scale.

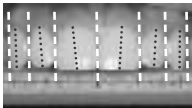Scale 2 contains the markings for the ideal average anterior tooth widths (vertical lines). This scale is used as a reference in the esthetic design of extensive anterior reconstructions. The markings present only relative values, from which the individual widths of the teeth can be derived or assessed in relation to each other. The oblique dotted markings indicate the typical average position of the tooth axis in the maxillary jaw and, consequently, also provide useful guidance in the reconstruction of harmonious anterior esthetics. The scale is positioned correctly if the triangle in the middle of the scale points towards the contact point between the central incisors and the numbers at both sides of the scale are consistent with the quadrants under examination. **The scale is unsuitable for use in the mandibular jaw!**

Scale 2 is utilized to compare the clinical situation against the ideal average tooth width proportions and angular alignments in the maxillary anterior dentition. Consequently, esthetic enhancements can be easily planned and areas of improvement identified.

Important! This scale indicates relative values only. The marked widths and angular alignments should not just be adopted as they are. They should be adjusted to meet the requirements of the individual situation.

The example depicted above shows comparatively ideal tooth width proportions, even if the overall tooth width may appear slightly larger than average. The angles of the second incisors and canines appear to be a touch too straight.

Warning

It is recommended to use a rubber dam to avoid accidental ingestion or aspiration of small parts.

Do not use OptraSculpt Pad for purposes other than intended!

Disinfection, cleaning and sterilization

General notes:

Clean and disinfect every instrument before each use. It is strongly recommended that the instruments are additionally sterilized in a steam sterilizer. Do not use damaged instruments. Only suitably trained staff should carry out repair and maintenance work.

Do not place the instruments in NaCl solution (to avoid the risk of pitting and/or stress corrosion cracking).

Protective gloves and goggles complying with the requirements of directive 89/686/EEC must be worn to handle any used and contaminated instruments. Disinfect and clean instruments immediately after use.

Generally, processing in a washer-disinfector unit is preferable to manual cleaning.

Pre-cleaning:

Carefully remove large unhardened residues from the instruments using a lint-free cellulose tissue. For preliminary disinfection, place the instruments in a disinfection bath (with a lid and sieve insert) filled with aldehyde-free alkaline disinfectant (e.g. immersion in ID 212 forte, Dürer Dental, for 15 min). Make sure that the instruments are completely covered in disinfectant and that the disinfectant is free of bubbles.

Only use a soft brush for the manual removal of residues. Never use a metal brush or steel wool.

For automated processing, carefully rinse the instruments under running tap water after preliminary disinfection.

Automated cleaning and disinfection:

- Washer-disinfector unit (e.g. Miele thermal washer-disinfector G7882)
- Cleaning and disinfection program (e.g. Vario TD)
- Detergent and disinfectant (e.g. Neodisher FA, Dr. Weigert)

If the instruments are only cleaned in a washer-disinfector, subsequent thermal disinfection in a steam sterilizer is mandatory.

Manual cleaning and disinfection:

- Soft brush (e.g. Miltex nylon cleaning brush for instruments)
- Detergent and disinfectant (e.g. immersion in ID 212 forte, Dürr Dental, for at least 15 min)

After cleaning and disinfection, thoroughly rinse the instruments in the sieve insert under running tap water.

Drying:

- Compressed air or clean, lint-free cellulose tissue

Sterilization:

- Suitable sterile packaging (e.g. aluminium sterile container system)
- Autoclave (e.g. Systec HX-320)
- Program: steam sterilization with a fractionated pre-vacuum process at 134 °C (273 °F) for at least 4 min.

Maintenance, checking and inspection:

After cleaning and disinfecting the instruments, visually inspect them to see if they are clean, undamaged and function properly. If macroscopically visible residual debris is present, repeat the cleaning and disinfection process.

Storage

- Store instruments in dry conditions to avoid condensation build-up.
- Store OptraSculpt Pad attachments in the original packaging at room temperature, protected from dust, moisture, compression and contamination.
- Store at 12 - 28 °C (54 - 82 °F).
- Expiry date: see printed inscription and label.

Notes

-  The handheld instrument is autoclavable and designed for multiple use and at least 200 sterilization cycles in the autoclave. See "Disinfection, cleaning and sterilization" for the stipulated procedures.
- Before autoclaving the handheld instrument, remove the modelling attachments.
-  The OptraSculpt Pad attachments are intended for single use only. They must not be cleaned, disinfected, sterilized and reused.
- Since small parts generally involve a risk of ingestion and aspiration, it is advisable to use a rubber dam.
-  The Instructions for Use and detailed information on the reprocessing of handheld instruments are available on the website of Ivoclar Vivadent AG (www.ivoclarvivadent.com).

Store out of the reach of children!

For dental use only!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia

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www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai

Trading Co., Ltd., 2/F Building 1, 881 Wuding Road, Jing An District

200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

www.ivoclarvivadent.co

Ivoclar Vivadent SAS

B.P. 118, F-74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2, D-73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

www.ivoclarvivadent.de

Wieland Dental + Technik

GmbH & Co. KG, Schwenninger Strasse 13, D-75179 Pforzheim, Germany

Tel. +49 7231 3705 0, Fax +49 7231 3579 59

www.wieland-dental.com

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, A, ndheri (West), Mumbai, 400 053, India

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www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.

Al. Jana Pawla II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

www.ivoclarvivadent.pl

Ivoclar Vivadent Marketing Ltd.

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

P.O. Box 300146, Riyadh 11372, Saudi Arabia

Tel. +966 11 293 8345, Fax +966 11 293 8344

www.ivoclarvivadent.com

Ivoclar Vivadent Pte. Ltd.

171 Chin Swee Road, #02-01 San Centre, Singapore 169877

Tel. +65 6535 6775, Fax +65 6535 4991

www.ivoclarvivadent.com

Ivoclar Vivadent S.L.U.

C/ Ribera del Loira nº 46, 5ª planta, 28042 Madrid, Spain

Tel. + 34 913 757 820, Fax + 34 913 757 838

www.ivoclarvivadent.es

Ivoclar Vivadent AB

Dalvägen 14, S-169 56 Solna, Sweden

Tel. +46 8 514 939 30, Fax +46 8 514 939 40

www.ivoclarvivadent.se

Ivoclar Vivadent Liaison Office

: Tesvikiye Mahallesi, Sakayik Sokak, Nisantas' Plaza No:38/2

Kat:5 Daire:24, 34021 Sisli – Istanbul, Turkey

Tel. +90 212 343 0802, Fax +90 212 343 0842

www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Ground Floor Compass Building, Feldspar Close, Warrens Business Park,

Enderby, Leicester LE19 4SE, United Kingdom

Tel. +44 116 284 7880, Fax +44 116 284 7881

www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive, Amherst, N.Y. 14228, USA

Tel. +1 800 533 6825, Fax +1 716 691 2285

www.ivoclarvivadent.us